



Abbott Diabetes Care Ltd
Range Road
Witney
Oxfordshire
OX29 0YL, UK

September 27th, 2020

To whom it may concern,

Disposal Guidelines for Sensors

- **FreeStyle Libre Flash Glucose Monitoring System Sensor and FreeStyle Libre 2 Flash Glucose Monitoring System Sensor**

Sensor:

Sensor must not be disposed of via municipal waste collection. Separate collection for electrical and electronic equipment waste as per Directive 2012/19/EC in the European Union is required. Contact the manufacturer for details. As Sensors may have been exposed to bodily fluids, you may wipe prior to disposing, such as by using a cloth dampened with a mixture of one part household bleach and nine parts water. **Note:** Sensors contain non-removeable batteries and must not be incinerated. Batteries may explode upon incineration.

Sensor Applicator:

Please consult your local waste management authority for instructions on how to dispose of Sensor Applicators at a designated sharps collection site. Ensure the cap is on the Sensor Applicator as it contains a needle.

Sensor Pack:

Used Sensor Packs may be disposed of via municipal waste collection.

- **FreeStyle Libre 3 Continuous Glucose Monitoring System Sensor**

Sensor:

Sensors must not be disposed of via municipal waste collection. Separate collection for electrical and electronic equipment waste per Directive 2012/19/EC in the European Union is required. Contact the manufacturer for details. As Sensors may have been exposed to bodily fluids, you may wipe prior to disposing, such as by using a cloth dampened with a mixture of 1 part household bleach to 9 parts water. **Note:** Sensors contain non-removable batteries and must not be incinerated. Batteries may explode upon incineration.

Sensor Applicator:

Please consult your local waste management authority for instructions on how to dispose of Sensor Applicators at a designated sharps collection site. Ensure the cap is on the Sensor Applicator as it contains a needle.

- **Abbott Libre Sense Glucose Sport Biosensor**

Biosensor:

The Biosensor must not be disposed of via standard waste collection. Separate collection for electrical and electronic equipment waste per Directive 2012/19/EU in the European Union is required. Contact the manufacturer for details. As Biosensors may have been exposed to bodily fluids, you may wipe prior to disposing, such as using a cloth dampened with a mixture of 1 part household bleach and 9 parts water.

Note: Biosensor contains one silver oxide non-removeable battery and must not be incinerated. Batteries may explode upon incineration.

Biosensor Applicator:


Please consult your local waste management authority for instructions on how to dispose Biosensor Applicators at a designated sharps collection site. Ensure the cap is on the Biosensor Applicator as it contains a needle.

Biosensor Pack:

Used Biosensor Packs may be disposed of via standard waste collection.

Sincerely,

Signed:

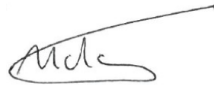

Printed Name:

Uday Bhagavatula

Title:

Senior Regulatory Affairs Specialist - EMEA, Abbott Diabetes Care

Date:

27 September 2021